


# QuikClot® INTERVENTIONAL® Hemostatic Bandage



QuikClot Interventional hemostatic bandage is indicated for the local management and control of surface bleeding from vascular access sites, percutaneous catheters or tubes utilizing introducer sheaths up to 12 Fr. or up to 7 Fr. for patients on drug/induced anti-coagulation treatment.



**1** Remove hemostatic pad from packaging and place pad into sterile field, using aseptic technique. Do not wet the pad with saline before using it.

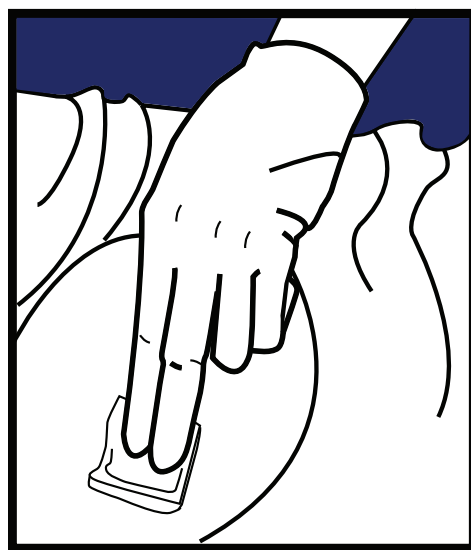


**2** Place the hemostatic pad on the puncture site. Apply manual compression on the pad for at least 5 minutes or until bleeding stops.

NOTE: Arterial punctures using large dilators will require longer manual compression time. Following placement of the hemostatic pad, health care professionals are encouraged to continue to use the standard of care at their institution regarding site care, time to ambulation and time to patient discharge.



**3** Without moving or lifting the pad, apply Tegaderm™ adhesive bandage or equivalent over pad while maintaining manual compression on the pad. Secure adhesive bandage to skin.

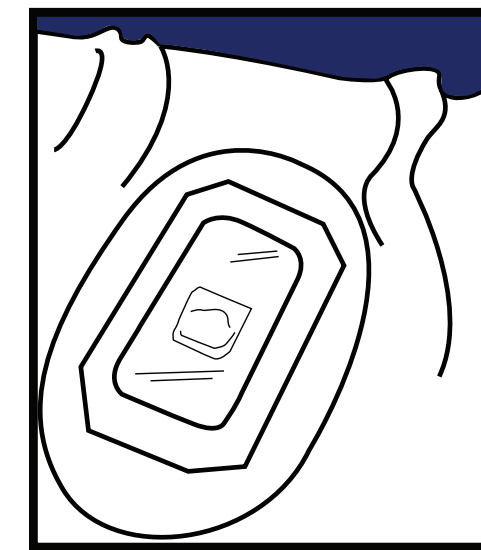

**4** The hemostatic pad should be changed every 24 hours or more often, if required. To change bandage, gently peel away adhesive bandage and gently remove pad.